

Item No.	Description	Packaging	
		Inner	Outer
VACUETTE® QUICKSHIELD Safety Tube Holder			
450230	VACUETTE® QUICKSHIELD Safety Tube Holder	50 pcs.	1000 pcs.
450238	VACUETTE® QUICKSHIELD Complete 21G x 1", sterile Safety Tube Holder & VACUETTE® Multiple Use Drawing Needle Pre-assembled Product	30 pcs.	300 pcs.
450237	VACUETTE® QUICKSHIELD Complete 21G x 1 1/2", sterile Safety Tube Holder & VACUETTE® Multiple Use Drawing Needle Pre-assembled Product	30 pcs.	300 pcs.
450234	VACUETTE® QUICKSHIELD Complete 22G x 1", sterile Safety Tube Holder & VACUETTE® VISIO PLUS Needle Pre-assembled Product	30 pcs.	300 pcs.
450235	VACUETTE® QUICKSHIELD Complete PLUS 21G x 1", sterile Safety Tube Holder & VACUETTE® VISIO PLUS Needle Pre-assembled Product	30 pcs.	300 pcs.
450239	VACUETTE® QUICKSHIELD Complete PLUS 21G x 1 1/2", sterile Safety Tube Holder & VACUETTE® VISIO PLUS Needle Pre-assembled Product	30 pcs.	300 pcs.
450226	VACUETTE® QUICKSHIELD Complete PLUS 22G x 1", sterile Safety Tube Holder & VACUETTE® VISIO PLUS Needle Pre-assembled Product	30 pcs.	300 pcs.
450228	VACUETTE® QUICKSHIELD Complete PLUS 22G x 1 1/2", sterile Safety Tube Holder & VACUETTE® VISIO PLUS Needle Pre-assembled Product	30 pcs.	300 pcs.
VACUETTE® Multiple Use Drawing Needles (sterile)			
450047	VACUETTE® Multiple Use Drawing Needle, pink 1.25 x 25mm 18G x 1"	100 pcs.	2000 pcs.
450048	VACUETTE® Multiple Use Drawing Needle, pink 1.25 x 38mm 18G x 1 1/2"	100 pcs.	2000 pcs.
450071	VACUETTE® Multiple Use Drawing Needle, black 0.7 x 25mm 22G x 1"	100 pcs.	2000 pcs.
450072	VACUETTE® Multiple Use Drawing Needle, green 0.8 x 25mm 21G x 1"	100 pcs.	2000 pcs.
450073	VACUETTE® Multiple Use Drawing Needle, yellow 0.9 x 25mm 20G x 1"	100 pcs.	2000 pcs.
450075	VACUETTE® Multiple Use Drawing Needle, black 0.7 x 38mm 22G x 1 1/2"	100 pcs.	2000 pcs.
450076	VACUETTE® Multiple Use Drawing Needle, green 0.8 x 38mm 21G x 1 1/2"	100 pcs.	2000 pcs.
450077	VACUETTE® Multiple Use Drawing Needle, yellow 0.9 x 38mm 20G x 1 1/2"	100 pcs.	2000 pcs.
VACUETTE® VISIO PLUS Needles (sterile)			
450040	VACUETTE® VISIO PLUS Needle, green 0.8 x 38mm 21G x 1 1/2"	100 pcs.	2000 pcs.
450041	VACUETTE® VISIO PLUS Needle, black 0.7 x 38mm 22G x 1 1/2"	100 pcs.	2000 pcs.
450042	VACUETTE® VISIO PLUS Needle, green 0.8 x 25mm 21G x 1"	100 pcs.	2000 pcs.
450043	VACUETTE® VISIO PLUS Needle, black 0.7 x 25mm 22G x 1"	100 pcs.	2000 pcs.

These devices have no components made of natural dry rubber.

Your Power for Health

greiner bio-one



Your Power for Health

greiner bio-one



For further information, please visit our website www.gbo.com/preanalytics or contact us:

Austria (Main Office)
Greiner Bio-One GmbH
Phone (+43) 75 83 67 91-0
Fax (+43) 75 83 63 18
E-Mail office@at.gbo.com

France
Greiner Bio-One SAS
Phone (+33) 1 69 86 25 25
Fax (+33) 1 69 86 25 92
E-Mail office@fr.gbo.com

Netherlands
Greiner Bio-One B.V.
Phone (+31) 1 72 42 09 00
Fax (+31) 1 72 44 38 01
E-Mail info@nl.gbo.com

Thailand
Greiner Bio-One Thailand Ltd
Phone (+66) 3 84 65 63-30
Fax (+66) 3 84 65 63-6
E-Mail office@th.gbo.com

Brazil
Greiner Bio-One Brasil
Phone (+55) 19 34 68 96 00
Fax (+55) 19 34 68 96 21
E-Mail office@br.gbo.com

Germany
Greiner Bio-One GmbH
Phone (+49) 201 861 86-0
Fax (+49) 201 861 86-12
E-Mail office@de.gbo.com

Spain
VACUETTE Espana S.A.
Phone (+34) 91 652 77 07
Fax (+34) 91 652 33 35
E-Mail info@vacuette.es

UK
Greiner Bio-One Ltd.
Phone (+44) 14 53 82 52 55
Fax (+44) 14 53 82 62 66
E-Mail info@uk.gbo.com

Egypt
Greiner Bio-One Middle East
Phone (+20) 26 21 87 06
Fax (+20) 26 21 87 08
E-Mail hisham.ouda@gbo.com

Hungary
Greiner Bio-One Hungary Kft.
Phone (+36) 96 21 30 88
Fax (+36) 96 21 31 98
E-Mail office@hu.gbo.com

Switzerland
Greiner Bio-One VACUETTE Schweiz GmbH
Phone (+41) 7 12 28 55 22
Fax (+41) 7 12 28 55 21
E-Mail office@ch.gbo.com

USA
Greiner Bio-One North America Inc.
Phone (+1) 70 42 61 78 00
Fax (+1) 70 42 61 78 99
E-Mail office@us.gbo.com

980053 rev. 04_01.2011 e



**VACUETTE®
QUICKSHIELD
Safety Tube Holder**

Our Innovations
for Your Safety

VACUETTE® QUICKSHIELD Safety Tube Holder

The **VACUETTE® QUICKSHIELD** Safety Tube Holder is especially suitable for use in daily blood collection routines. Without the need to change collection techniques, the holder helps prevent needlestick injuries.



VACUETTE® QUICKSHIELD Protective Cap

Is pre-assembled directly on the holder. The cap is activated immediately upon withdrawal of the needle from the patient's vein.

Item No. 450230

VACUETTE® QUICKSHIELD Safety Tube Holder
With ergonomic design for improved comfort.

Activated with the aid of a solid surface and/or thumb.

Comfort through one-handed activation.

User-friendly - dot on needle container allows user to choose which side the **VACUETTE® QUICKSHIELD** Safety Tube Holder should be positioned for optimal venipuncture.

Exclusively for use with **VACUETTE®** Multiple Use Drawing Needles or **VACUETTE®** VISIO PLUS Needles.


No changes to collection technique.

Especially safe as the fingers remain behind the needle tip at all times.

The protective cap is pre-attached to the holder, remaining stable. Once activated, there is no chance of the protective cap coming off.

An audible click indicates to the user that the protective cap has been fully activated.

VACUETTE® Multiple Use Drawing Needles

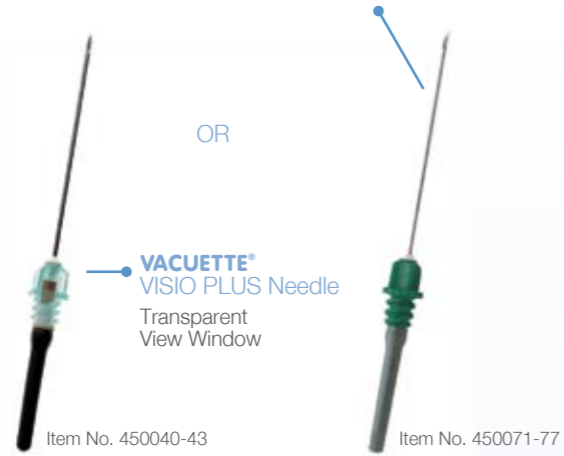

OR

VACUETTE® VISIO PLUS Needle Transparent View Window

Item No. 450040-43

Item No. 450071-77

For pain-free venipuncture. The perfectly matched needle and holder mean that the needle tip is always oriented in the correct position for easy specimen collection.

VACUETTE® QUICKSHIELD Combination Products

Simply Practical.
The **VACUETTE® QUICKSHIELD** Combination Products for Blood Collection.

VACUETTE® QUICKSHIELD Complete (Safety Tube Holder & **VACUETTE®** Multiple Use Drawing Needle Combination Product)

- The holder comes with pre-attached needle, sterile and single-packed.
- It is not necessary to thread in the needle.
- A practical, ready-to-use solution that saves time.
- Handling is optimised.



Item No. 450234, 450237-38

VACUETTE® QUICKSHIELD Complete PLUS (Safety Tube Holder & **VACUETTE®** VISIO PLUS Needle Combination Product)

- The holder comes with pre-attached needle, sterile and single-packed.
- Transparent view window for optical venipuncture control.
- It is not necessary to thread in the needle.
- A practical, ready-to-use solution that saves time.
- Handling is optimised.




Item No. 450226, 450228,
450235, 450239

Instructions for Use *



1** Thread the needle

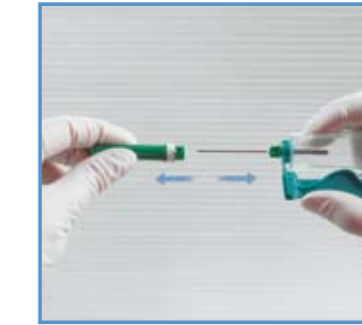

2 Remove protective cap



3 Venipuncture



4 Insert the tube



5 Remove the last tube



6 Activate the safety mechanism



7 Audible click indicates activation



8 Disposal

* For the complete blood collection procedure please refer to the Instructions for Use.

** This step is not applicable for QUICKSHIELD Complete & Complete PLUS.